




Innovation without boundaries



# LiNA AntiFog Solution™

*Provides clear camera optics during endoscopic surgery*



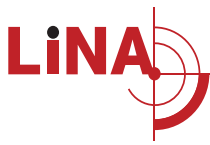

# LiNA AntiFog Solution™

LiNA Medical's range of antifog solutions consists of LiNA Elfred, LiNA Clear Sight and LiNA Clear Sight Wipe.

## functional

- **LiNA Elfred**

LiNA Elfred consists of a small sponge in an aluminium foil container with adhesive on the back. The sponge is prefilled with 6ml antifog solution. LiNA Elfred offers a high degree of convenience as it is simple and ready for use:

- 1) Remove the adhesive strip under the container and attach it to the patient drape or instrument table.
- 2) Remove the lid of the container.
- 3) Press the scope onto the sponge and twist it to apply a membrane of the antifog solution.
- 4) If required, the sponge can be taken out of the container to clean the sides of the scope.

The LiNA Elfred is available with or without alcohol. The alcohol version has a longer shelf life.

- **LiNA Clear Sight**

LiNA Clear Sight consists of a 6ml bottle and a sponge with adhesive on the back. LiNA Clear Sight is easy to use:

- 1) Attach the sponge to the patient drape or instrument table.
- 2) Open the bottle and pour solution onto the sponge.
- 3) Press the scope onto the sponge and turn it to apply a membrane of the antifog solution.

The LiNA ClearSight is available with or without alcohol. The alcohol version has a longer shelf life.

- **LiNA Clear Sight Wipe**

LiNA Clear Sight Wipe is prefilled with antifog solution and individually packed in a peel pouch. The LiNA Clear Sight Wipe is only available without alcohol.



LiNA Clear Sight Wipe



LiNA Elfred



LiNA Clear Sight

### Ordering Information:

LiNA AntiFog Solutions **CE** 0459

REF.	Product	Packing
CS-20	LiNA Elfred - standard, single use.	Box of 30, sterile
CS-20A	LiNA Elfred - with 4% alcohol, single use.	Box of 30, sterile
CS-30	LiNA Clear Sight - standard, single use.	Box of 20, sterile
CS-30A	LiNA Clear Sight - with 4% alcohol, single use.	Box of 20, sterile
CS-40	LiNA Clear Sight Wipe – standard, single use.	Box of 30, sterile



LiNA Medical ApS

• Formervangen 5  
DK-2600 Glostrup  
Denmark

• Tel.: +45 43 29 66 66  
Fax: +45 43 29 66 99  
info@lina-medical.com

• www.lina-medical.com  
CVR: 10054974