

The Mount Business Park
2 Woodstock Link
Belfast BT6 8DD
Northern Ireland

T: +44 (0)28 9073 7283
F: +44 (0)28 9073 7282
sales@haemobandsurgical.com
www.haemobandsurgical.com




The **multi-band ligator** that's simple, fast, reliable and ready to use.



Having been designed by surgeons for surgeons, the Haemoband makes ligation procedures more comfortable for patients, more clinically efficient for surgeons and easier for nurses.

H1001

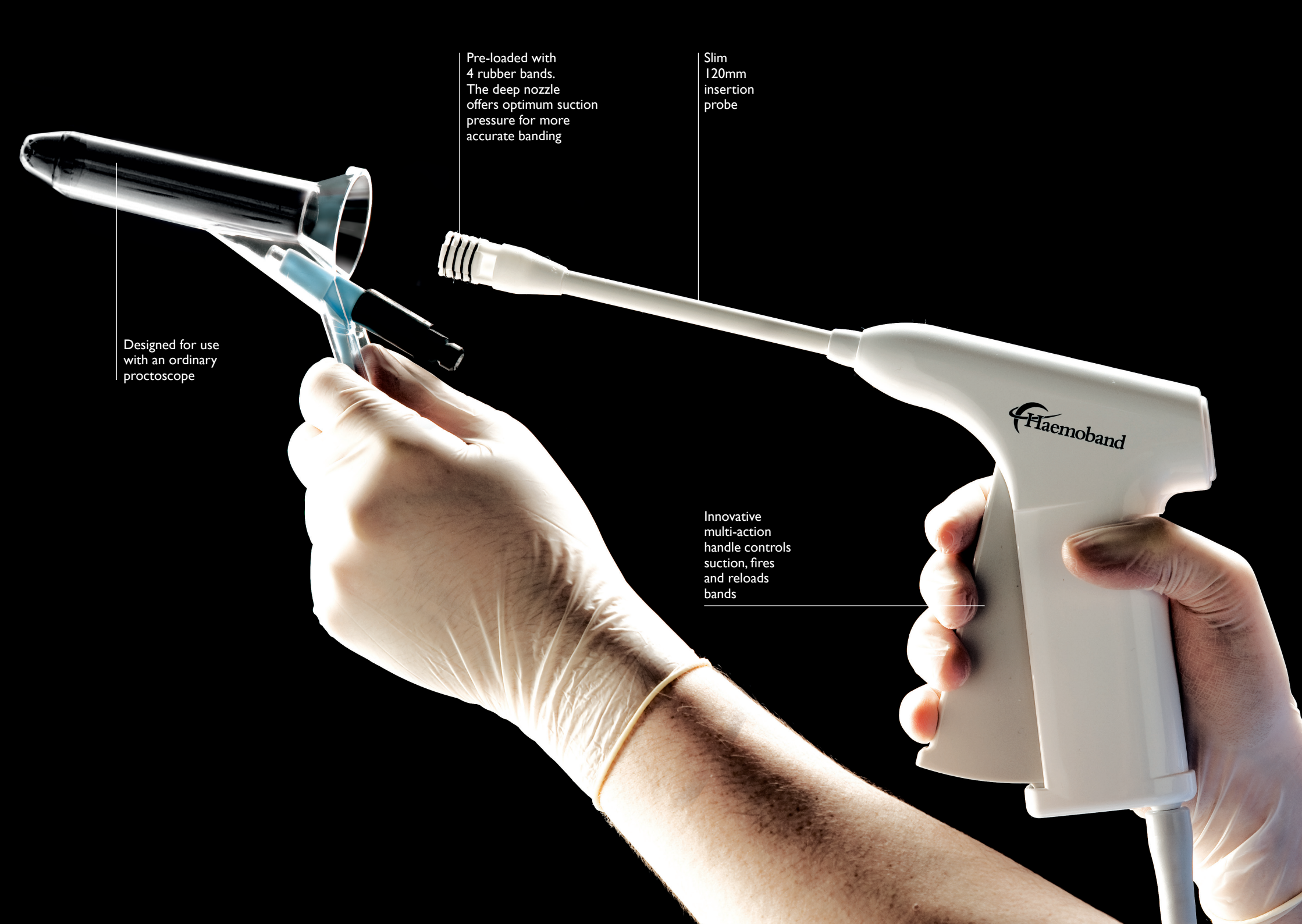

Designed for use with an ordinary proctoscope

Pre-loaded with 4 rubber bands. The deep nozzle offers optimum suction pressure for more accurate banding

Slim 120mm insertion probe

Innovative multi-action handle controls suction, fires and reloads bands

A better, more clinically efficient ligator for Patients, Nurses and Surgeons

Clinically more efficient

Haemoband is very clinically efficient. Its multi-action handle means you only need one hand to control suction and release and reload bands. This leaves your other hand free to hold a proctoscope and means you can work alone. The design of Haemoband also allows for much more accurate applications, reducing the need for additional procedures.

Easier and faster


Because the gun is pre-loaded, already clean and can be used single-handed, the time taken for a typical ligation procedure can be reduced from 20 minutes to between 2 and 4 minutes for the banding of up to 3 haemorrhoids. This not only improves patient comfort considerably, but its disposability negates the risk of cross-contamination.

Ready to use

The Haemoband gun is ready to use straight from the airtight, clean bag. There are no spare parts to lose, find or fit and the only extra equipment you need is a proctoscope and standard hospital suction.

Cost effective

The Haemoband is one of the most cost effective multi-ligators on the market. Its disposability means that there is no need for sterilisation equipment and no need to invest in costly permanent tools.

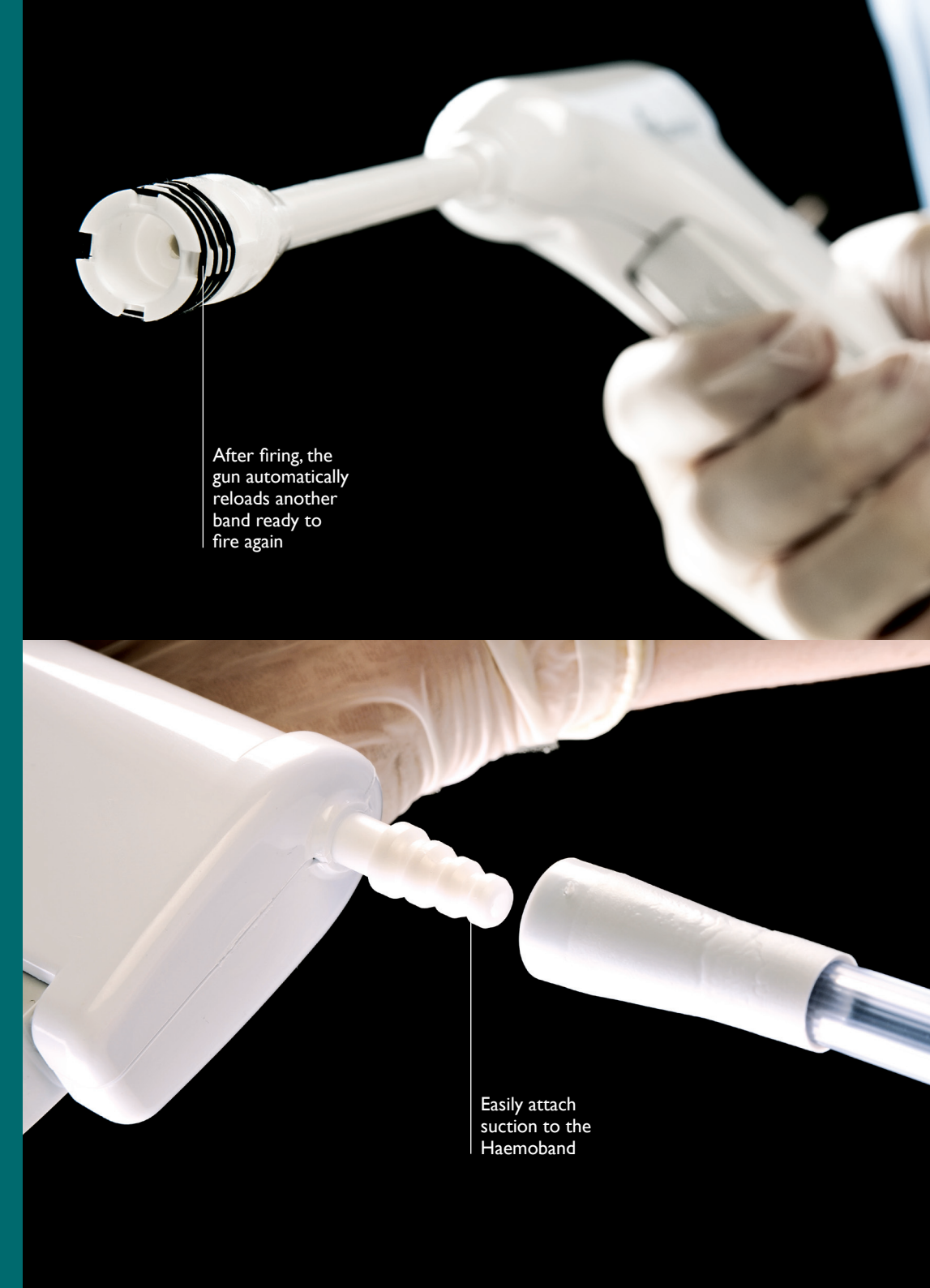

After firing, the gun automatically reloads another band ready to fire again

Easily attach suction to the Haemoband